STIFNECK®

CE

Original Extrication Collar and
Select™ Adjustable Extrication Collar

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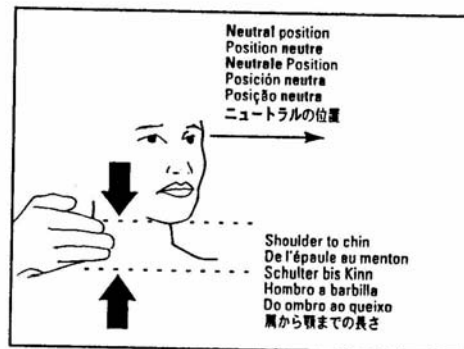
Align the head to neutral or "eyes forward" position unless contraindicated by your protocol.

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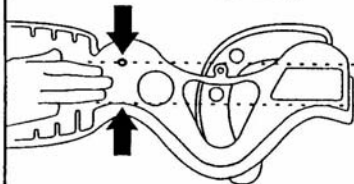
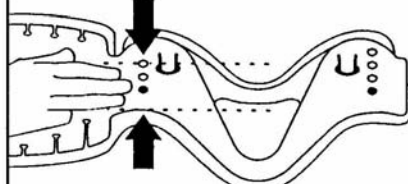
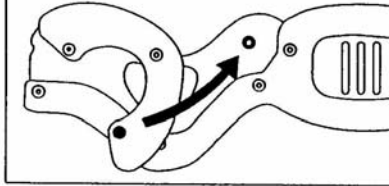
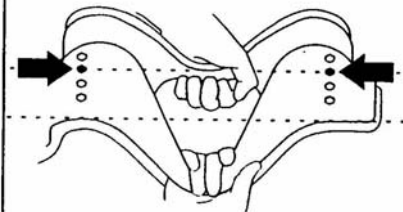
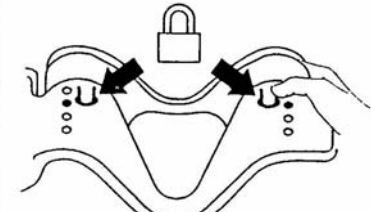
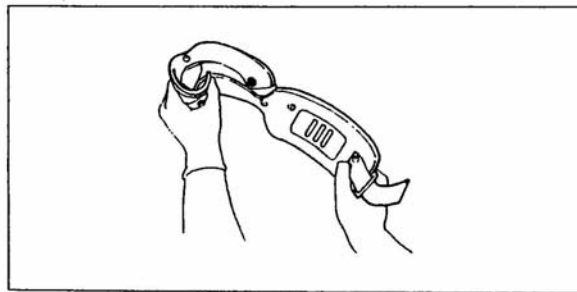
Place the chin support well under the chin. If a different size is needed, remove, re-size, and re-apply the collar.

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Pull the back of the collar snug while holding the front in place, then fasten.

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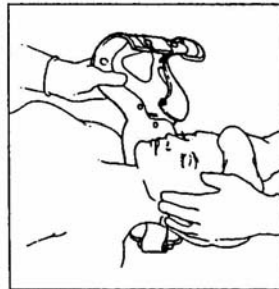
For a supine patient, slide the rear panel behind the neck, before placing the chin support.

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Important: Do not adjust the Select collar on patient.

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Storage: Do not store collar in folded position. Store Flat.

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IMPORTANT

Do not rely on a collar alone to properly restrict the motion of a patient's cervical spine.

Do not use an improperly sized collar. Too large a collar may hyperextend a patient's cervical spine; too small a collar may provide appropriate stability. Special sizes of Stinneck collars are available for children and others with small bodies.

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WARRANTY

LIMITED WARRANTY

Laerdal Medical Corporation, 187 Myers Corners Road, Waukegan Falls, NY 12990 8848, warrants to the user who originally purchases this product that it is free from defects in material and workmanship. The sole obligation of Laerdal Medical Corporation in the event of breach of warranty shall be to repair or replace the defective product or part(s).

Laerdal Medical Corporation shall have no obligation under this limited warranty in the event:

- (a) The product was not purchased from Laerdal Medical Corporation or through its authorized channels of distribution.
- (b) The product is altered.
- (c) Any parts not supplied by Laerdal Medical Corporation are inserted into the product or.
- (d) The product is not used in accordance with Laerdal Medical Corporation instructions for use.

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THE FOREGOING IS THE SOLE AND EXCLUSIVE REMEDY FOR FAILURE IN SERVICE OF OR DEFECT IN THE PRODUCT. LAERDAL MEDICAL CORPORATION SHALL NOT BE LIABLE UNDER THIS OR ANY IMPLIED WARRANTY FOR ANY DIRECT, SPECIAL, INCIDENTAL, OR CONSEQUENTIAL DAMAGES. THIS WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, EXPRESS OR IMPLIED, INCLUDING THE WARRANTY OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE OR USE. AND ALL OBLIGATIONS OR WARRANTIES OF THE PART OF LAERDAL MEDICAL CORPORATION FOR DAMAGES ARISING OUT OF OR IN CONNECTION WITH THE USE OF THE PRODUCTS, WHICH ARE HEREBY DISCLAIMED AND EXCLUDED BY LAERDAL MEDICAL CORPORATION.

This warranty gives you certain legal rights, and you may have other rights which vary from state to state.

Warning: To insure proper use, please review all material.

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Laerdal
helping save lives

Manufactured in USA by:
Laerdal Medical Corporation
187 Myers Corners Road
Waukegan Falls, NY 12990-8840
(800) 431-1055 or (845) 297-7770

Distributed by:
Laerdal Medical Norway
P.O. Box 377, N-4001 Stavanger
Tel: 47 51 51 17 00, Fax 51 52 35 57
Laerdal Medical Canada, Ltd.
151 Nashdene Rd., Unit #45
Toronto, ON, Canada, M1V 4C3
Tel: (416) 298-9600,
Toll free 888/LAERDAL (523-7325)
ou en français (800) 567-9987

Intended for single patient use.
À utiliser que sur un seul patient.
Nur vom Einzelpatienten Gebrauch bestimmt.
Destinado para uso em um único paciente.
Para uso exclusivo por um único paciente.
患者一人への使用に限定

The product is in compliance with the essential requirements of Council Directive 93/42/EEC Medical Devices Directive.

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U.S. Patent No. 32,219 U.S. Patent No. 8,293,718
Anders Patents and Inventions
© 2001 Laerdal Medical Corporation. Alle Rechte vorbehalten.

Patente de EE.UU. n° 32.219 Patente de EE.UU. n° 8.293.718
Brevets patents en français
© 2001 Laerdal Medical Corporation. Todos los derechos reservados.

本国特許
© 2001 Laerdal Medical Corporation. Alle Rechte vorbehalten.

© 2001 Laerdal Medical Corporation. All Rights Reserved.

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© 2001 Laerdal Medical Corporation. Tous droits réservés.